

How to

Download CoreGateway

Completing these steps prior to your modality configuration will help expedite the appointment.

Before Installation:


1. Administrators will have access to download CoreGateway by default. Other users can be granted permission to download CoreGateway from the Download CG permission in the User Permissions settings.
2. You will need a windows computer, and administrative access on the PC. Please discuss with your IT team who has those permissions.

Step 1: Select the **CG Download** button on the top of your Studies page.




Step 2: Click **Download**.

Follow the steps to complete the installation and save the installer to your computer

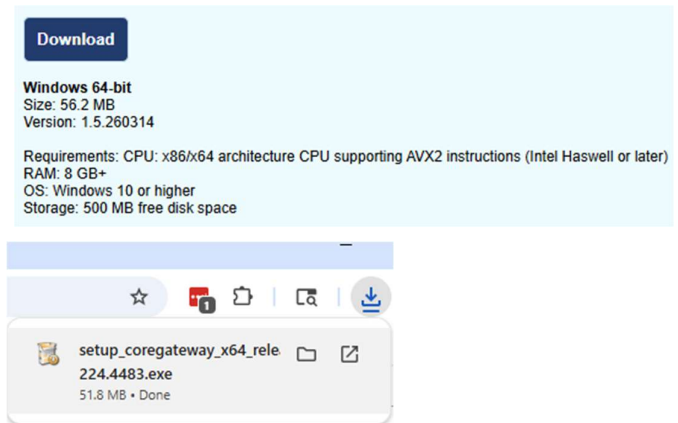


Step 3: Continue moving through the installation process by clicking **Next**, accepting the license terms, and install CoreGateway.

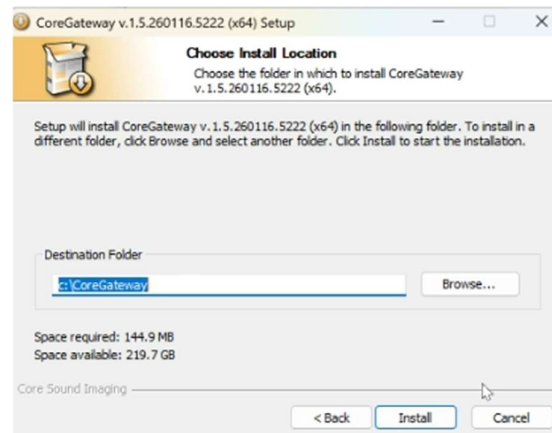

Step 4. Ensure **Enable automatic updates** is selected.

Click **Configure this CoreGateway in StudyCast** and select the division to which these studies should be uploaded.

Note: See step 5 on how to select divisions.

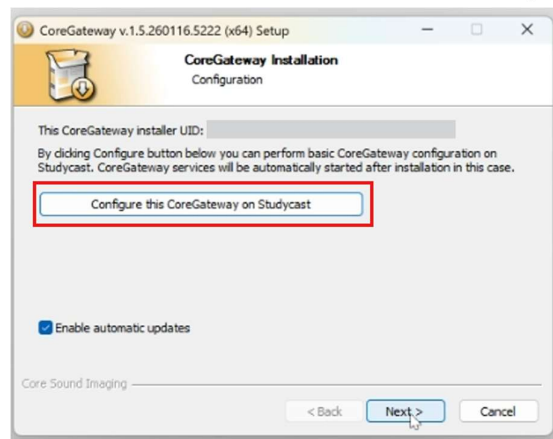

Step 5: Once you selected **Configure this CoreGateway on StudyCast**, it will open a new browser tab where you can select the specific division to which this CoreGateway will upload studies. Once you've chosen your division, select **Configure**.



Step 6: Navigate back to the installer and click finish to complete the installation.

Note: your browser may be covering the installer on your screen when you configured divisions

